
Mobile Stand for Tourniquet

Intended to be left blank.

	PRODUCT DESCRIPTION	4	HR	STATIV ZA TOURNIQUET	18
	4		Upute za uporabu	18
	SCOPE OF DELIVERY	4	HU	MOBIL ÁLLVÁNY TOURNIQUET KÉSZÜLÉKEKHEZ	19
	4		Használati utasítás	19
	ASSEMBLY INSTRUCTION MOBILE STAND	5 - 6	IT	SUPPORTO MOBILE PER TOURNIQUET	20
	5 - 6		Istruzioni per l'uso.....	20
	TRANSPORT CONDITIONS	6	LT	VARŽČIO STOVAS	21
	6		Naudojimo instrukcija.....	21
	ASSEMBLY INSTRUCTION ACCESSORY BASKET (OPTIONAL)	7	LV	STATĪVS TOURNIQUET IERĪCĒM	22
	7		Lietošanas instrukcija	22
DE	STATIV FÜR TOURNIQUET	8	NL	STATIEF VOOR TOURNIQUET	23
	Gebrauchsanweisung	8		Gebruiksaanwijzing.....	23
EN	MOBILE STAND FOR TOURNIQUET	9	NO	STATIV FOR TOURNIQUET	24
	Instructions for use.....	9		Bruksanvisning.....	24
BG	СТОЙКА ЗА ТУРНИКЕТ	10	PL	STATYW DO TOURNIQUET	25
	Инструкции за употреба	10		Instrukcja użycia.....	25
CS	STATIV NA TOURNIQUET	11	PT	SUPORTE PARA TOURNIQUET	26
	Návod k použití.....	11		Instruções de utilização	26
DA	STATIV TIL TOURNIQUET	12	RO	STATIV PENTRU TOURNIQUET	27
	Bruganvisning.....	12		Instrucțiuni de utilizare	27
EL	ΚΥΛΙΟΜΕΝΗ ΒΑΣΗ ΓΙΑ ΣΥΣΚΕΥΗ ΙΣΧΑΙΜΗΣ ΠΕΡΙΔΕΣΗΣ	13	RU	ШТАТИВ ДЛЯ ПРИБОРА TOURNIQUET	28
	Οδηγίες χρήσης.....	13		Инструкция по применению	28
ES	SOPORTE PARA TOURNIQUET	14	SK	STATÍV PRE TOURNIQUET	29
	Instrucciones de uso.....	14		Návod na použitie	29
ET	TOURNIQUET' STATIIV	15	SL	STOJALO ZA NAPRAVE TOURNIQUET	30
	Kasutamisujuhised.....	15		Navodila za uporabo	30
FI	TOURNIQUET-LAITTEEN JALUSTA	16	SV	STATIV FÖR TOURNIQUET	31
	Käyttöohje	16		Bruksanvisning.....	31
FR	SUPPORT MOBILE POUR TOURNIQUET	17	TR	TOURNIQUET İÇİN STAND	32
	Manuel d'utilisation	17		Kullanım kılavuzu	32
				SYMBOL DESCRIPTION	33 - 35
				33 - 35

PRODUCT DESCRIPTION

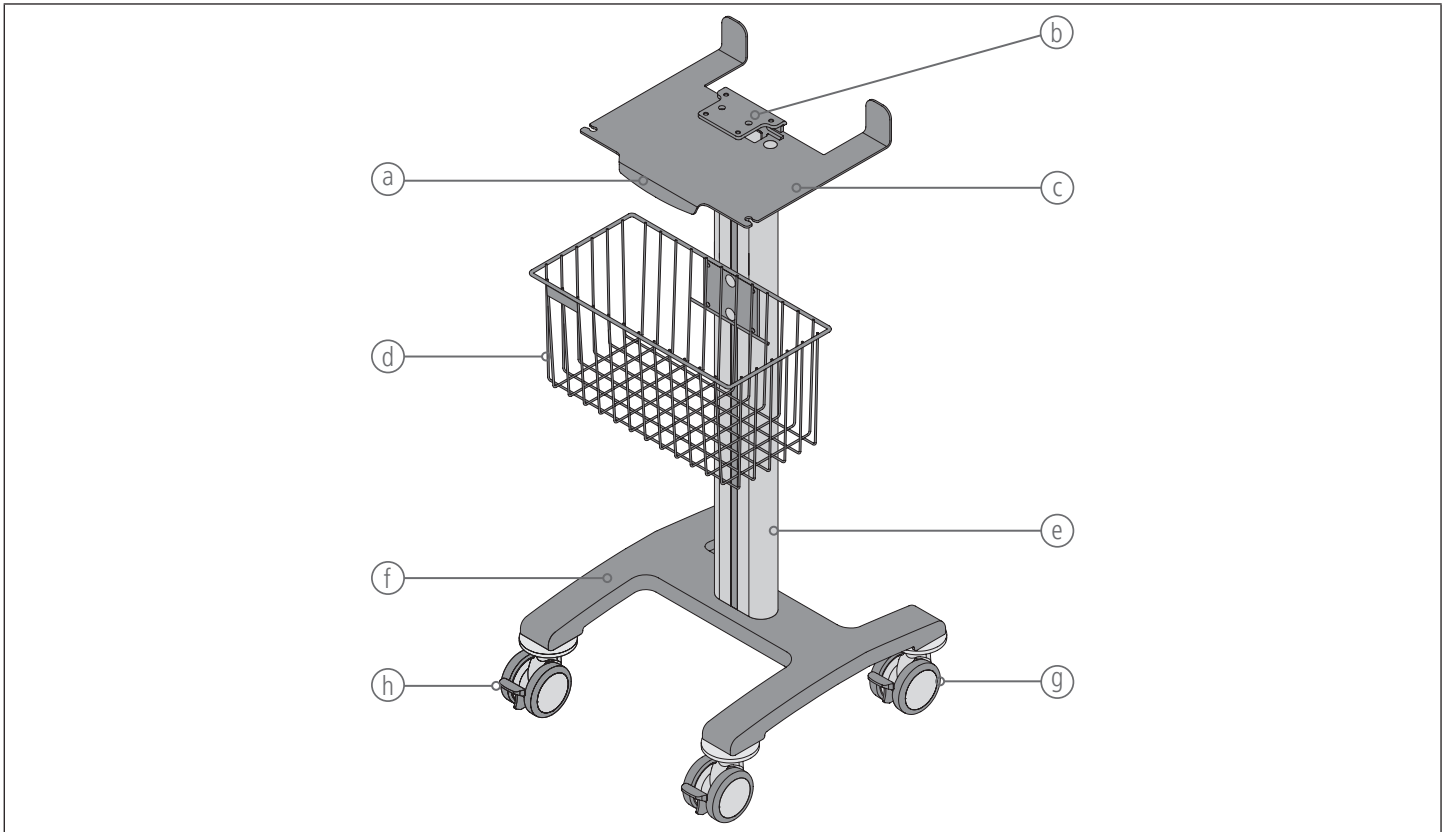SCOPE OF DELIVERY

REF 01-00-100 Mobile Stand for Tourniquet

A 1x 	B 1x 	C 1x 	D 1x 	E 2x (M6) Form S	F 2x 6.3 x 65 mm	G 1x
H 2x M6x10	I 2x 	J 4x 	K 1x 	L 2x M3x16	M 2x 6.3 x 25 mm	N 2x M5x8
O 2x Ø 5.3 mm	P 1x Size 4	Q 1x Size 2.5	R 1x 	S 4x M6x16		

REF 01-00-110 Basket for Mobile Stand

T 1x 	U 2x M6x10	V 2x 	W 2x 	X 1x Size 4	Y 1x
-----------------	--------------------------	-----------------	-----------------	---------------------------	-----------------

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English

INTENDED USE

Transport and supply of Tourniquet Touch devices.

Place of use: Rooms which are suitable for surgical interventions.

INDICATIONS / CONTRAINDICATIONS

Indications / contraindications are determined by the Tourniquet Touch device.

SAFETY INSTRUCTIONS

- Read and follow the instructions for use carefully before using the product and keep them for future reference.

- This product must only be used by medically trained personnel.
- The user and/or patient must report all serious adverse events that occurred in connection with the product to the manufacturer and competent authorities of the EU member state (or report to the competent authorities of the country if an event occurs outside of the EU) in which the user and/or patient is located.
- Before every use, the product must be inspected visually for damages (cracks, breakage, etc.). A faulty product must not be used.
- It is not permitted to make any changes to the product.
- After each use, the product must be disinfected (see "Wipe disinfection").
- The device mounted on the bracket must not weigh more than 5 kg (11 lbs).
- The basket may be loaded with a maximum of 2 kg (4.4 lbs).
- The loaded Mobile Stand can tilt over when the maximum weight is exceeded.
- The product is not sterile.
- The product is not suitable for MRI.

PRODUCT DESCRIPTION (SEE PAGE 4)

- | | |
|-------------|-----------------|
| (a) Handle | (e) Column |
| (b) Bracket | (f) Foot-stand |
| (c) Plate | (g) Castor |
| (d) Basket | (h) Castor lock |

USE

CAUTION

- Do not lean against, sit on or step onto the product!
- The Mobile Stand can tilt over on a ramp > 5°.
- During transport of the Mobile Stand, the "Transport conditions" section must be observed.

- To lock the castors, push down **all** castor locks.

CAUTION

- To prevent the Mobile Stand from moving, lock **all** castors with the castor locks. If not all castors are locked, the Mobile Stand can move unintentionally.
- The Mobile Stand can tilt over if it is pushed or pulled with locked castors.

- To unlock the castors, push up **all** castor locks.

ASSEMBLY INSTRUCTIONS FOR MOBILE STAND (SEE PAGES 5 - 6)

- Attach the foot-stand (A) to the column (B) (figure 1). The rating plate and the warning and prohibition signs on the column must face the short side of the foot-stand and the plate.
- Attach the basket (G) to the column (B) (figure 2).
To do this, loosely fasten both countersunk screws to the basket plate with the Allen key (H) and the slot nuts (I). Then hang the basket (G) into the groove of the column and tighten the countersunk screws. Place the protective caps (J) on the countersunk screws. The top of the basket must be located at a height of approx. 71 cm (28"). The height is specified by the groove cover.

NOTE

A second basket (T) can be attached optionally. This is optionally available from the manufacturer. To assemble the second basket, see the "Assembly instructions for accessory basket (optional)" section.

- Pull the mains cable through the column (B) so that the V-Lock inlet connector for non-heating devices protrudes approx. 28 cm (11") from the column (figure 3).
- To ensure strain relief, attach the mains cable to the foot-stand using the strain relief sleeve (K) (figure 4).
- Attach the plate (D) to the column (B). Place the protective caps (J) on the countersunk screws (figure 5).
- Install the bracket onto the Tourniquet Touch device (figure 6).
- Assemble the bracket onto the column (B) (figure 7).
- Connect the V-Lock inlet connector for non-heating devices to the Tourniquet Touch device (figure 8).
- Clip the groove cover (C / 200 mm (8")) into the groove and press lightly (figure 9).

ASSEMBLY INSTRUCTIONS FOR ACCESSORY BASKET (OPTIONAL, SEE PAGE 7)

Assemble the accessory basket as follows:

- Disconnect the mains cable from the Tourniquet Touch device.
- Place the Mobile Stand down on the floor. Remove the basket which is already assembled and the groove covers (figure 12).
- Attach the groove cover (Y / 104 mm (4")) (figure 13).
- Insert the slot nuts (V). Assemble the second basket (figure 14). The top of the second basket must be located at a height of approx. 33 cm (13"). The height is specified by the groove cover.
- Attach the groove cover (Y / 300 mm (11¾")) (figure 15).
- Assemble the first basket (figure 16). The top of the first basket must be located at a height of approx. 71 cm (28"). The height is specified by the groove cover.

- Attach the groove cover (C / 200 mm (8")) (figure 17).
- Place the Mobile Stand upright again.
- Connect the mains cable to the Tourniquet Touch device.

WIPE DISINFECTION

CAUTION

- The product must not be mechanically reprocessed or sterilised.
- Do not immerse the product in liquid or clean with a wet cloth.

Wipe disinfection must be performed with commercially available surface disinfectants based on alcohol or QACs (quaternary ammonium compounds). When choosing a product for disinfection, a disinfectant with appropriate ranges of action must be used: bactericidal, yeasticidal and virucidal. After wipe disinfection, inspect the product for visible contamination. If necessary, repeat wipe disinfection.

SHELF LIFE

The shelf life of the product is 8 years.

ENVIRONMENTAL CONDITIONS

CAUTION

- Protect from heat and store in a dry place.
- Keep away from sunlight and light sources.

Environmental conditions for transport:

Temperature: -30 °C to +60 °C (-22 °F to 140 °F)

Humidity: 5 to 95 % relative humidity, non-condensing

Environmental conditions for storage / operation:

Temperature: +10 °C to +35 °C (+50 °F to +95 °F)

Humidity: 30 to 95 % relative humidity, non-condensing

TRANSPORT CONDITIONS (SEE PAGE 6)

CAUTION

- To prevent slipping or tilting of the Mobile Stand during transport, the specified transport conditions must be observed!
- If the following instructions are not followed, personal injury or damage to property could result.

The Mobile Stand with mounted Tourniquet Touch device may only be transported under the following conditions:

- The mains cable must be attached to the plate behind the Tourniquet Touch device (figure 10).
- The loading of the basket must be arranged evenly.
- The baskets must not be loaded beyond their top level.
- The connecting tubings of the Tourniquet Touch device must be attached to the lateral openings of the plate (figure 11).
- Move the Mobile Stand only by the handle (a).
- To prevent the Mobile Stand from moving, lock **all** castors with the castor locks. If not **all** castors are locked, the Mobile Stand can move unintentionally.

DISPOSAL

The faulty product must be disposed of in accordance with the applicable national and international regulations.

PRODUCT SPECIFICATIONS

REF	01-00-100	01-00-110
Description	Mobile Stand for Tourniquet	Basket for Mobile Stand
Weight	8.2 kg (18.1 lbs)	0.73 kg (1.6 lbs)
Dimensions	Height 939 mm (37") / width 400 mm (15¾") / depth 400 mm (15¾")	Height 185 mm (7¼") / width 377 mm (14¾") / depth 196 mm (7¾")

- The maximum acceptable total weight of 18.6 kg (41 lbs) must not be exceeded.
- The device mounted on the bracket must not weigh more than 5 kg (11 lbs).
- The basket may be loaded with a maximum of 2 kg (4.4 lbs).

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