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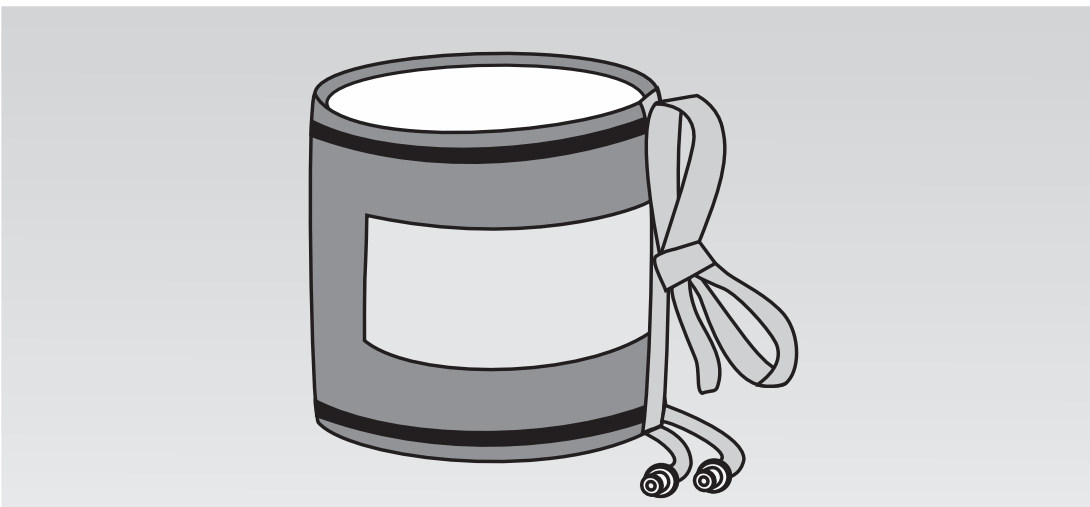
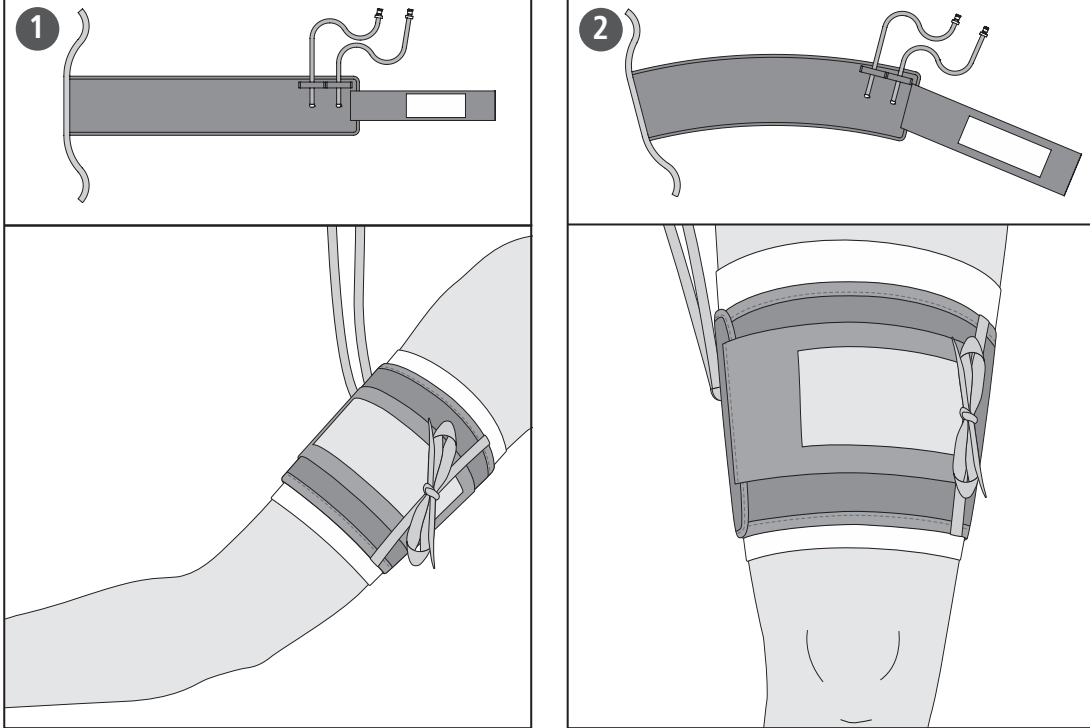
Tourniquet Dispo Cuff

DE	TOURNIQUET DISPO CUFF MIT ZWISCHLAUCHSYSTEM Gebrauchsanweisung	LT	TOURNIQUET DISPO CUFF SU DVIJŲ ŽARNŲ SISTEMA Naudojimo instrukcija.
EN	TOURNIQUET DISPO CUFF WITH DUAL PORT Instructions for use	LV	TOURNIQUET DISPO CUFF MAŅŠĒTE AR DIVU CAURULIŅU SISTĒMU Instrukcija par lietošanu.
BG	TOURNIQUET DISPO CUFF СЪС СИСТЕМА С ДВА МАРКУЧА Инструкции за употреба	NL	TOURNIQUET DISPO CUFF MET TWEESLANGENSTELSTEM Gebrauchsanwijding
CS	TOURNIQUET DISPO CUFF SE SYSTĚMOM DVOU HADIČEK Návod k použití	PL	TOURNIQUET DISPO CUFF Z SYSTEMEM DWUPRZEWODOWYM Instrukcja użytko
DA	TOURNIQUET DISPO CUFF MED TOSLANGESYSTEM Brugsanvisning	PT	TOURNIQUET DISPO CUFF COM SISTEMA DE DOIS TUBOS Instruções de utilização
EL	TOURNIQUET DISPO CUFF ME SYΣTHMA ΔYO EYKAMITON ΣΔΑΗΝΗ Οδηγίες χρήσης	RO	TOURNIQUET DISPO CUFF CU SISTEM CU DOUA FURTUNURI Instrucțiuni de utilizare
ES	TOURNIQUET DISPO CUFF CON SISTEMA DE DOS TUBOS Instrucciones de uso	RU	КРОВОСТАНАВЛЮВАЮЩАЯ МАКМЕТКА ТУРНИКЕТ ДИСПО CUFF С ДВУХШАХЛЮВНОЙ СИСТЕМОЙ Инструкция по использованию
ET	KÄHETOURSISTEEMIGA TOURNIQUET DISPO CUFF MANSETT Kasutamishüts	SK	MAKMETKA TOURNIQUET DISPO CUFF S DVOJHAČOVÝM SYSTĚMOM Návod na použitie
FI	TOURNIQUET DISPO CUFF -MANSETTI KAKSILETKÄJÄRIESEN Käyttöohje	SL	MAKMETKA TOURNIQUET DISPO CUFF Z DVOECVNIM SISTEMOM Navodila za uporabo
FR	TOURNIQUET DISPO CUFF AVEC SYSTÈME À DOUBLE TUBULURE Manuel d'utilisation	SV	TOURNIQUET DISPO CUFF MED TVÅ SLANGAR Instruktioner för användning
HR	TOURNIQUET DISPO CUFF SA SUSTAVOM S DVA CRIEVA Upute za uporabu	TR	İKİ HORTUM SİSTEMLİ TOURNIQUET DISPO CUFF Kullanım kılavuzu
HU	TOURNIQUET DISPO CUFF KÉTŐMŐLŐS RENDSZERREL Használati utasítás	IT	TOURNIQUET DISPO CUFF CON SISTEMA A DOPPIO TUBO Istruzioni per l'uso

	004-01-0337 Rev. 03/2022-04
---	-----------------------------

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EN **TOURNIQUET DISPO CUFF WITH DUAL PORT**
Instructions for use

INTENDED USE – TOURNIQUET DISPO CUFF
Tourniquet Dispo Cuffs are tourniquet cuffs which are used during surgical procedures to ensure a temporary bloodless field in the patient's upper or lower extremities.

INTENDED USE - EXTENSION TUBING
The extension tubing allows the sterile zone of the Tourniquet Dispo Cuff to be extended.

LOCATION OF USE
Rooms that are suitable for surgical procedures.

INDICATION / CONTRAINDICATION
Indications and contraindications depend on the application and therefore on the selected Dispo Cuff.

Possible indications for blood arrest:

- Rectification of certain fractures
- Arthroscopy of knee, hand, finger or elbow
- Bone grafting
- Kirschner wire removal
- Traumatic or non-traumatic amputations
- Removal of tumours or cysts
- Subcutaneous fasciotomy
- Nerve damage
- Ligament repair
- Replacement or revision of knee joint, wrist or finger joint
- Correction of a hammer toe
- Podiatry

No other indications are known.

Possible contraindications for blood arrest:

- Open fractures of the extremities
- Post-traumatic lengthy hand reconstruction
- Severe crushing injuries
- Elbow surgery with concomitant excessive swelling
- Severe hypertension
- Skin transplant
- Impaired circulation (e.g. peripheral arterial disease)
- Diabetes mellitus

No other contraindications are known. The physician must, in each individual case, assess the indications and contraindications in the light of his or her expert knowledge.

SAFETY INSTRUCTIONS

- Read the instructions for use carefully before using the product and follow them.
- This product must only be used by a physician, or by medically trained personnel working under the instruction of a physician.
- The user must wear gloves when using the product.
- The user and/or the patient must report all serious adverse events that have occurred in connection with the product to the manufacturer and to the competent authority of the EU member state in which the user and/or patient is practising (see section 4 for the competent authority of the country concerned if an event occurs outside of the EU).
- Prior to each use, the products must be subjected to a visual inspection (see section "Visual inspection"). Damaged products must not be used.
- Maximum application time: 2 hours
- It is not permitted to make any changes to the product.
- The Tourniquet Dispo Cuffs are designed for use with Tourniquets with dual ports.
- The Dispo Cuff is designed for a max. cuff pressure of 600 mmHg when in place on the patient. If exposed to a higher cuff pressure, the Dispo Cuff will be irreversibly damaged.
- The product is intended for single use and must not be reused and/or reprocessed. The function of the product is impaired by reprocessing. Any reuse entails a potential infection hazard.
- The product is sterile (ethylene oxide).
- The product must not be used if the packaging is damaged or the expiration date has expired.

PREPARATION PRIOR TO USE
Select the correct cuff size for the respective extremity.

VISUAL INSPECTION

- Check the packaging for damages.
- Check the product for damages (cracks, breaks, kinks and loose particles).

Damaged products must not be used.

APPLICATION

CAUTION

- The duration of the blood arrest is at the physician's discretion. A max. of 2 hours is normally recommended.
- In order to ensure reliable blood arrest while avoiding exposure of the patient to unnecessary stress, an appropriate cuff target pressure must be selected for the Dispo Cuff, depending on the cuff size, the extremity and the systolic blood pressure.
- The user must check the current pressure of the Dispo Cuff at regular intervals. If the target pressure deviates from the current pressure of the Dispo Cuff, then appropriate action must be taken in response.
- Avoid kinking of cuff tubes.
- All tube connections must snap firmly into place.

- Apply thin padding under the Dispo Cuff to avoid pressure sores and injury to the skin.

In order to ensure that the cuff tubes are not located in the operating field, they must point proximalward (pictures 1 + 2).

CAUTION

To ensure reliable blood arrest, the bladder of the Dispo Cuff must overlap on the extremity. However, too much overlap can result in cuff rolling.

- Apply Dispo Cuff tightly on the extremity, but without excess force. Two fingers should fit between the extremity and the Dispo Cuff.
- After applying the Dispo Cuff, additionally secure it with the colour-coded cuff ties (pictures 1+ 2).

Option: The sterile zone of the Dispo Cuff can be extended by means of the extension tubing. Connect the extension tubing to the cuff tube.

- Connect the cuff tube or extension tubing to the Tourniquet.
- All tube connections must snap firmly into place.
- Prior to cuff inflation, the extremity must be exsanguinated.
- Inflate the cuff with the lowest necessary pressure required to occlude the blood flow.
- In order to avoid possible chemical burns, fluids must be prevented from getting under the Dispo Cuff.
- Disinfect the skin and drape the operating field.
- Before commencing the surgical procedure, make sure by means of palpation or auscultation that the arterial blood flow has been occluded.

CAUTION

- After use, completely deflate the Dispo Cuff and immediately remove cuff and padding from the extremity in order to prevent the risk of venous congestion.

- Disconnect the cuff tube from the Tourniquet.

SHelf LIFE
Expiration date: See label on the product.

STORAGe AND TRANSPORT CONDITIONS

CAUTION

- Protect from heat and store in a dry place.
- Keep away from sunlight and light sources.
- Store and transport in the original packaging.

DISPOSAL
The used or damaged products must be disposed of in accordance with the applicable national and international statutory regulations.

REF	Tourniquet Dispo Cuff
20-36-7005LZ-1	Single Cuff, 20 cm (8 inch)
20-36-7105LZ-1	Single Cuff, 30 cm (12 inch)
20-36-7115LZ-1	Single Cuff, 35 cm (14 inch)
20-36-7125LZ-1	Single Cuff, 46 cm (18 inch)
20-36-7155LZ-1	Single Cuff, 46 cm (18 inch)
20-36-7225LZ-1	Single Cuff, 61 cm (24 inch)
20-36-7252LZ-1	Single Cuff, 76 cm (30 inch)
20-36-7285LZ-1	Single Cuff, 86 cm (34 inch)
20-36-7295LZ-1	Single Cuff, 107 cm (42 inch)
20-39-100-1	Extension tubing for Tourniquet Dispo Cuffs, length 100 cm (40 inch)

REF	Width	Colour coding	Shape	Weight (incl. packaging)	Packaging unit
20-36-7005LZ-1	2 cm (2 inch)	-	-	1,06 kg (2,3 lbs)	1
20-36-7105LZ-1	6 cm (2 1/2 inch)	blue	straight	1,22 kg (2,7 lbs)	1
20-36-7115LZ-1	8 cm (3 1/8 inch)	yellow	-	1,40 kg (3,1 lbs)	1
20-36-7125LZ-1	10 cm (4 inch)	red	-	1,63 kg (3,6 lbs)	1
20-36-7155LZ-1	10 cm	blue	-	2,47 kg (5,5 lbs)	10
20-36-7225LZ-1	12 cm	green	-	2,69 kg (5,9 lbs)	10
20-36-7252LZ-1	12 cm (4 inch)	dark blue	tapered	2,86 kg (6,3 lbs)	10
20-36-7285LZ-1	12 cm	brown	-	3,55 kg (7,8 lbs)	10
20-36-7295LZ-1	12 cm	dark red	-	3,95 kg (8,7 lbs)	5

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