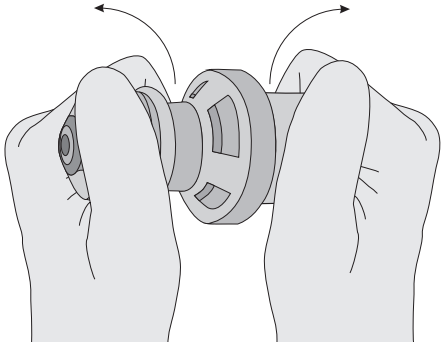
PEEP-Valve

single use

ASSEMBLY Page 3	LT	PEEP VOŽTUVAS, VIENKARTINIS NAUDOJIMAS Naudojimo instrukcija..... 17 psl.
DE	PEEP-VENTIL, EINMALGEBRAUCH Gebrauchsanweisung Seite 4	LV	PEEP VĀRSTS, VIENREIZLIETOJAMĀIS Lietošanas instrukcija 18. lpp.
EN	PEEP-VALVE, SINGLE USE Instructions for use..... Page 5	NL	PEEP-VENTIEL, VOOR ÉÉNMALIG GEBRUIK Gebruiksaanwijzing..... pagina 19
BG	PEEP ВЕНТИЛ, ЗА ЕДНОКРАТНА УПОТРЕБА Инструкции за употреба страница 6	NO	PEEP-VENTIL, ENGANGSBRUK Bruksanvisning..... Side 20
CS	VENTIL PEEP, JEDNORÁZOVÉ POUŽITÍ Návod k použití Strana 7	PL	ZAWÓR PEEP DO JEDNORAZOWEGO UŻYCIA Instrukcja użycia..... Strona 21
DA	PEEP-VENTIL, ENGANGSBRUG Brugsanvisning..... Side 8	PT	VALVULA PEEP, DESCARTÁVEL Instruções de utilização Páginas 22
EL	ΒΑΛΒΙΔΑ PEEP, ΜΙΑΣ ΧΡΗΣΗΣ Οδηγίες χρήσης..... Σελίδα 9	RO	VALVĂ PEEP, PRODUS DE UNICĂ FOLOSINȚĂ Instrucțiuni de utilizare pagina 23
ES	VALVULA PEEP, UN SOLO USO Instrucciones de uso..... Página 10	RU	PEEP-КЛАПАН ОДНОРАЗОВОГО ИСПОЛЬЗОВАНИЯ Инструкция по применению Страница 24
ET	PEEP KLAPP, ÜHEKORDESELT KASUTAMISEKS Kasutamissjuhís Lehekülj 11	SK	VENTIL PEEP, JEDNORAZOVÉ POUŽITIE Návod na použitie Strana 25
FI	PEEP-VENTTIILI, KERTAKÄYTTÖINEN Käyttöohje Sivu 12	SL	VENTIL PEEP, ZA ENKRATNO UPORABO Navodila za uporabo Stran 26
FR	VALVE DE PEP, À USAGE UNIQUE Mode d'emploi..... Page 13	SV	PEEP-VENTIL FÖR ENGÅNGSBRUK Bruksanvisning..... Sid. 27
HR	PEEP VENTIL, ZA JEDNOKRATNU UPORABU Upute za uporabu stranica 14	TR	PEEP-VALFİ, TEK KULLANIMLIK Kullanma kılavuzu Sayfa 28
HU	PEEP SZELEP, EGYSZER HASZNÁLATOS Használati utasítás 15. oldal		SYMBOL DESCRIPTION Page 29 - 31
IT	VALVOLA PEEP, MONOUSO Istruzioni per l'uso..... Pagina 16		

ASSEMBLY

PEEP-VALVE

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English

INTENDED USE

The PEEP-Valve is intended to generate a positive end-expiratory pressure.

INDICATION

Can prevent a collapse of the alveoli and thus prevent atelectasis.

CONTRAINDICATION

None known.

SAFETY INFORMATION

- Read and follow the instructions for use carefully before using the product.
- Intended for use by qualified personnel.
- Prior to use, a functional check must be performed (see section "Functional check"). Damaged products must not be used.
- Only use the PEEP-Valve if the patient's condition can be precisely monitored with blood gas analyses and haemodynamic measurements.
- The use of the PEEP-Valve on resuscitator bags and ventilation systems is only permitted if its use has been explicitly specified by the manufacturer.
- Do not use the product in a toxic atmosphere.
- Do not use oil, grease or any hydrocarbon-based substance on the product so as to avoid possible flammability during use.
- The product is intended for single use and must not be reprocessed. Reprocessing influences the function of the product. Any reuse entails a potential risk of cross-contamination.
- The product must not be used if the expiration date has expired.
- Use original VBM parts only. Using parts from other manufacturers may have a negative impact on the function of the medical device. The manufacturer accepts no liability for this.

FUNCTIONAL CHECK

- ▶ Check products for damages (cracks, breakages, etc.) and loose particles.
 - ▶ Test the PEEP-Valve (2 - 10 cmH₂O) at 5 and 10 cmH₂O.
 - ▶ Test the PEEP-Valve (5 - 20 cmH₂O) at 10 and 20 cmH₂O.
- The tolerance is ± 2 cmH₂O, and the values must be within this tolerance for 3 seconds.
- ▶ Damaged products must not be reused.

USE

- ▶ To change the lower housing, disassemble the PEEP-Valve in accordance with the diagram (see section "Assembly").
- ▶ Then reassemble the PEEP-Valve with the other lower housing in accordance with the diagram (see section "Assembly").
- ▶ Connect the PEEP-Valve (using the PEEP-Adapter if necessary) and set the desired PEEP-level by turning the adjustment knob on the PEEP-Valve.

SHELF LIFE

Expiration date: See product label

STORAGE AND TRANSPORT CONDITIONS

Operating temperature: -18 °C ... +50 °C (-0.4 °F ... 122 °F)

Storage temperature: -40 °C ... +60 °C (-40 °F ... 140 °F)

- Protect from heat and store in a dry place.
- Keep away from sunlight and light sources.
- Store and transport in the original packaging.

DISPOSAL

The used or damaged products must be disposed of in accordance with the applicable national and international statutory regulations.

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