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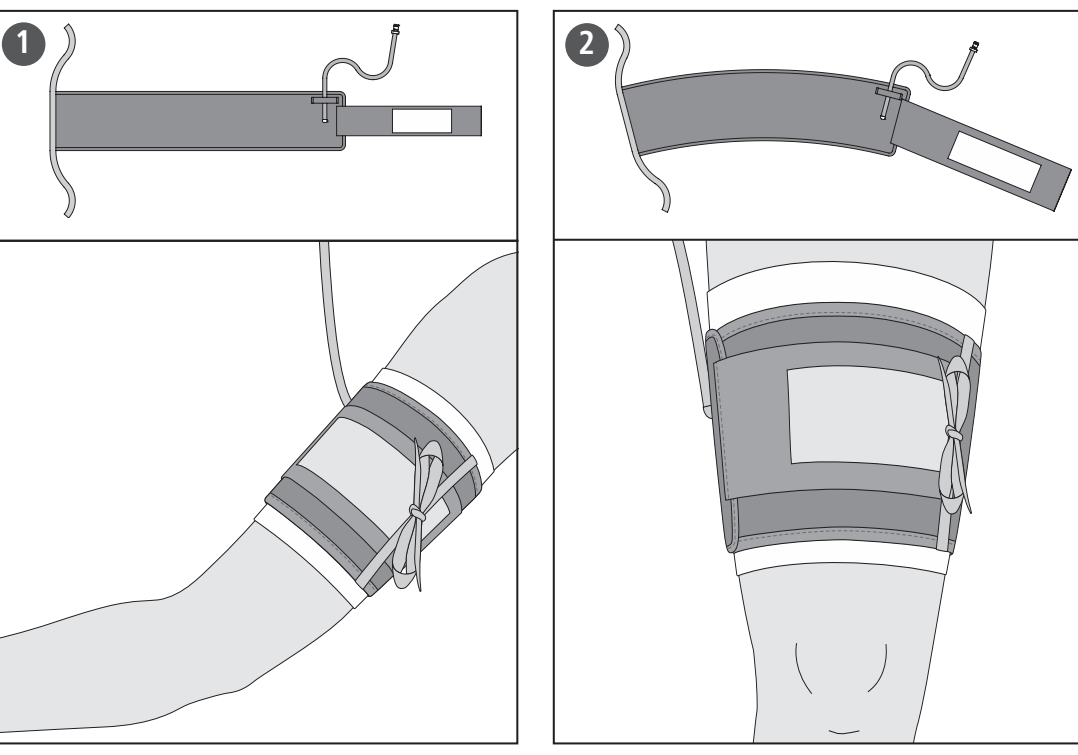
## Tourniquet Dispo Cuff

<b>DE</b>	<b>TOURNIQUET DISPO CUFF</b> Kasovazna instrukcia
<b>EN</b>	<b>TOURNIQUET DISPO CUFF</b> Instructions for use
<b>BG</b>	<b>ТОУРНИКЕТ ДИСПО ЦИФ</b> Инструкции за употреба
<b>CS</b>	<b>TOURNIQUET DISPO CUFF</b> Návod k použití
<b>DA</b>	<b>TOURNIQUET DISPO CUFF</b> Brugsanvisning
<b>EL</b>	<b>ΤΟΥΡΝΙΚΕΤ ΔΙΣΠΟ ΤΣΙΦ</b> Οδηγίες χρήσης
<b>ES</b>	<b>TOURNIQUET DISPO CUFF</b> Instrucciones de uso
<b>ET</b>	<b>TOURNIQUET DISPO CUFF</b> Kasvajamisjuhised
<b>FI</b>	<b>TOURNIQUET DISPO CUFF</b> Käyttöohje
<b>FR</b>	<b>TOURNIQUET DISPO CUFF</b> Manuel d'utilisation
<b>HR</b>	<b>TOURNIQUET DISPO CUFF</b> Upute za uporabu
<b>HU</b>	<b>TOURNIQUET DISPO CUFF</b> Használati utasítás
<b>IT</b>	<b>TOURNIQUET DISPO CUFF</b> Istruzioni per l'uso

<b>LT</b>	<b>TOURNIQUET DISPO CUFF</b> Kasovazna instrukcija
<b>LV</b>	<b>TOURNIQUET DISPO CUFF</b> Lietotājam instrukcija
<b>NL</b>	<b>TOURNIQUET DISPO CUFF</b> Gebruiksaanwijzing
<b>NO</b>	<b>TOURNIQUET DISPO CUFF</b> Bruksanvisning
<b>PL</b>	<b>TOURNIQUET DISPO CUFF</b> Instrukcja
<b>PT</b>	<b>TOURNIQUET DISPO CUFF</b> Instruções de utilização
<b>RO</b>	<b>TOURNIQUET DISPO CUFF</b> Instrucțiuni de utilizare
<b>RU</b>	<b>TOURNIQUET DISPO CUFF</b> Инструкция по использованию
<b>SK</b>	<b>TOURNIQUET DISPO CUFF</b> Návod na použitie
<b>SL</b>	<b>TOURNIQUET DISPO CUFF</b> Navodila za uporabo
<b>SV</b>	<b>TOURNIQUET DISPO CUFF</b> Bruksanvisning
<b>TR</b>	<b>TOURNIQUET DISPO CUFF</b> Kullanma talimatları
<b>SYMBOL DESCRIPTION</b>	

G1033 Rev. 19/2022-04

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**EN TOURNIQUET DISPO CUFF**  
Instructions for use

**INTENDED USE - TOURNIQUET DISPO CUFF**  
Tourniquet Dispo Cuffs are tourniquet cuffs which are used during surgical procedures to ensure a temporary bloodless field in the patient's upper or lower extremities.

**INTENDED USE - EXTENSION TUBING**  
The extension tubing allows the sterile zone of the Tourniquet Dispo Cuff to be located off-site.

**LOCATION OF USE**  
Rooms that are suitable for surgical procedures.

**INDICATION / CONTRAINDICATION**  
Indications and contraindications depend on the application and therefore on the selected Dispo Cuff.

**Possible indications for blood arrest:**

- Traumatic amputations
- Arthroscopy of knee, hand, finger or elbow
- Bone grafting
- Kirschner wire removal
- Removal of certain traumatic amputations
- Removal of tumours or cysts
- Subcutaneous fasciotomy
- Nerve damage
- Ligament repair
- Replacement or revision of knee joint, wrist or finger joint
- Correction of a hammer toe
- Podiatry

**Contraindications are known:**

- Open fractures of the extremities
- Post-traumatic lengthy hand reconstruction
- Severe crushing injuries
- Wound surgery concomitant excessive swelling
- Diabetes mellitus
- Spinal transplants
- Impaired circulation (e.g. peripheral arterial disease)

**Other contraindications are known:**

- Open fractures of the extremities
- Post-traumatic lengthy hand reconstruction
- Severe crushing injuries
- Wound surgery concomitant excessive swelling
- Diabetes mellitus
- Spinal transplants
- Impaired circulation (e.g. peripheral arterial disease)

No other contraindications are known. The physician must, in each individual case, assess the indications and contraindications in the light of his or her expert knowledge.

**SAFETY INSTRUCTIONS**

- Read the instructions for use carefully before using the product and follow them.
- This product must only be used by a physician, or by medically trained personnel working under the instruction of a physician.
- The user must wear gloves when using the product.
- The user and/or the patient must report all serious adverse events that have occurred in connection with the product to the manufacturer and to the competent authority of the EU member state in which the user and/or patient is practising/residing (or the competent authority of the country concerned if an event occurs outside of the EU).
- Prior to each use, the products must be subjected to a visual inspection (see section "Visual inspection"). Damaged products must not be used.
- Maximum application time: 2 hours
- It is not permitted to make any changes to the product.
- The Dispo Cuff has been designed and tested for use with Tourniquets from the manufacturer. If the user uses a Tourniquet from other manufacturers, then the manufacturer accepts no liability for the product.
- The Dispo Cuff is designed for a max. cuff pressure of 600 mmHg when in place on the patient. If exposed to higher cuff pressure, the Dispo Cuff will be irreversibly damaged.
- The product is intended for single use and must not be reused and/or repaired. The function of the product is impaired by processing. Any reuse entails a potential infection hazard.
- The product is sterile (ethylene oxide).
- The product must not be used if the packaging is damaged or the expiration date has expired.

**PREPARATION PRIOR TO USE**  
Select the correct cuff type for the respective extremity.

**VISUAL INSPECTION**

- Check the packaging for damages (cracks, breaks, kinks and loose particles).
- Damaged products must not be used.

**APPLICATION**

**CAUTION**

- The duration of the blood arrest is at the physician's discretion. A max. of 2 hours is maximumly recommended.
- In order to ensure reliable blood arrest while avoiding exposure of the patient to unnecessary stress, an appropriate cuff target pressure must be selected for the Dispo Cuff, depending on the cuff size, the extremity and the systolic blood pressure.
- The user must check the target pressure regularly at the Dispo Cuff at regular intervals. If the target pressure deviates from the current pressure of the Dispo Cuff, then appropriate action must be taken in response.
- Avoid kinking of cuff tubes.
- All tube connections must snap firmly into place.

Apply this padding under the Dispo Cuff to avoid pressure sores and injury to the skin.

In order to ensure that the cuff tubes are not located in the operating field, they must point posteriorward (pictures 1 + 2).

**CAUTION**

- To ensure reliable blood arrest, the bladder of the Dispo Cuff must overlap on the extremity. However, too much overlap can result in cuff rolling.
- Apply Dispo Cuff tightly on the extremity, but without using force. Two fingers should fit between the extremity and the Dispo Cuff.
- After applying the Dispo Cuff, additionally secure it with the colour-coded cuff ties (pictures 1 + 2).

Option: The sterile zone of the Dispo Cuff can be extended by means of the extension tubing. Connect the extension tubing to the cuff tube.

**None contraindications**

- Connect the cuff tube to extension tubing.
- All tube connections must snap firmly into place.
- Prior to cuff inflation the extremity must be tourniqueted.
- Inflate the cuff with the lowest necessary pressure required to occlude the blood flow.
- In order to avoid possible chemical burns, fluids must be prevented from getting under the Dispo Cuff.
- Disinfect the skin and drape the operating field.
- Diabetes commencing the surgical procedure must be sure by means of palpation or auscultation that the arterial blood flow has been occluded.

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