Mobile Stand with Basket

Spare Part for VBM Tourniquet 2500

DE	MONTAGE Stativ	Seite 3 - 4
EN	ASSEMBLY Mobile stand	Page 5 - 6
ES	MONTAJE Pie movable	Página 7 - 8
FR	MONTAGE Pied roulant	Page 9 - 10
IT	MONTAGGIO Stativo mobile	Pagina 11 - 12
NL	MONTAGE Statief	Pagina 13 - 14
SV	MONTERING Ställning	Sida 15 - 16
	ASSEMBLING OF FIXATION SCREWS	Page 17
	SYMBOL DESCRIPTION	Page 18

1. Montage Stativ

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

1. Assembly of mobile stand

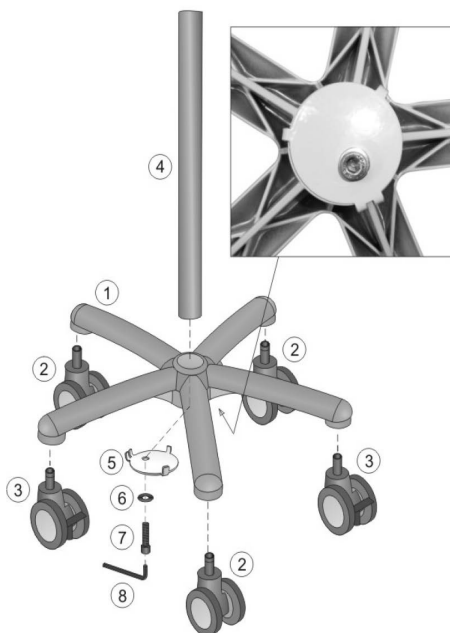- 1.1 Attach the lock-type rollers (3) opposite of each other to the base (1).
- 1.2 Attach the ordinary rollers (2) to the base (1).
- 1.3 Insert the support rod (4) into the base (1).
- 1.4 Insert cylinder hat screw M8x40 (7) with washer (6) and distortion lock (5) from below into the base (1). Fasten with hexagon socket wrench (8).

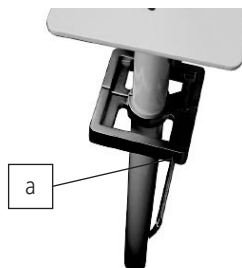
- 1.5 Fix the square rail by means of the two hexagon socket screws to the support rod (a).

2. Assembly of fixation plate

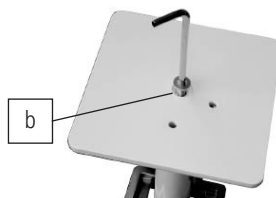- 2.1 Screw the fixation plate with counter sunk screw M8x20 to the support rod (b).
- 2.2 Turn the Tourniquet and attach the screws according to the drawing (see page 17).

- 2.3 Attach the device to the fixation plate.
- 2.4 Fix the device with nut M6 and wrench SW11.

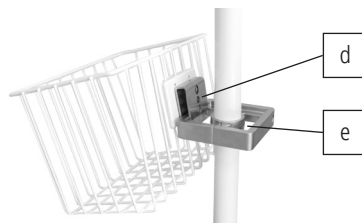
3. Assembly of basket

- 3.1 Attach the steel hook with fixation plate (c) to the basket using two countersunk screws M5x12.

- 3.2 Attach the basket to the square rail (e) with the steel hook (d).

4. Service

The product may only be serviced by persons who have the technical knowledge, qualification and the required means to properly carry out the service.

After service of the product, the essential constructive and functional characteristics for safety and functionality have to be checked, insofar as they can be influenced by the service procedure.

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