DE Wasserfalle

Seite 3 - 5

EN Watertrap

Page 6 - 8

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Página 9 - 11

FR Piège à eau

Page 12 - 14

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Pagina 15 - 17

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REPROCESSING (CLEANING, DISINFECTION, STERILISATION)

GENERAL INFORMATION

- Products are supplied non-sterile and must be unpacked, cleaned, disinfected and sterilised prior to first use.
- Products must be reprocessed prior to each use.
- Manufacturer is not responsible for damages resulting from improper reprocessing.
- It rests with the user to validate their procedures, devices and accessories, and to comply with the validated parameters during every reprocessing.
- It is recommended to apply an automatic procedure due to a significantly higher efficiency of cleaning and disinfection.
- The effectiveness has been verified by an independent and accredited test laboratory.
- To achieve a more effective reprocessing rough contamination must not get dry on the product and must be removed immediately after use.

CLEANING / DISINFECTION

Automatic Cleaning / Disinfection

Notes regarding the use of washer disinfectors

- Use de-ionized water.

Notes regarding the use of cleaning agents and disinfectants

- By the use of alkaline cleaning agents, neutralisation must be performed.
- Don't use drying agents.

Procedure validated by the manufacturer:

1. Disassemble the product, as shown in the drawing "Assembly".
2. Place the products in a suitable mesh tray into the injector carriage. Secure the products with a cover net. Ensure effective rinsing of all hollow spaces.
3. Start program with the following parameters:
 - a. Rinse with de-ionized water at 20 °C (68 °F) with an exposure time of 1 min.
 - b. Cleaning at 55 °C (131 °C) with an exposure time of 5 min using de-ionized water and the cleaner "Sekumatic® ProClean" (Dose: 0,5 % (5 ml/l)).
 - c. Neutralisation with "Sekumatic® FNZ" at 20 °C (68 °F) with an exposure time of 2 min (Dose: 0,1 % (1 ml/l)).
 - d. Rinse with de-ionized water at 20 °C (68 °F) and an exposure time of 2 min.
 - e. Thermal disinfection at 93 °C (199 °F) for 5 min with de-ionized water.
 - f. Drying at 100 °C (212 °F) for 10 min.
4. Examine the products for visible contamination. If necessary repeat the reprocessing.
5. All parts must be dry and free of damage (cracks, ruptures etc.) prior to assembly. Assemble the product, as shown in the drawing "Assembly". Prior to assembly lubricate the washer with e.g. silicone spray.

6. Check the products acc. to the chapter "Control" and prepare them for sterilisation (see chapter "Packaging").

Manual Cleaning / Disinfection

Notes regarding the use of cleaning agents and disinfectants

- Prepare the cleaning- and disinfection solution before each manual cycle.
- Use de-ionized water.

Procedure validated by the manufacturer:

1. Disassemble the product, as shown in the drawing "Assembly".
2. Prepare a 2 % (30 ml/l) cleaning- and disinfection solution with "Sekusept® Aktiv" and de-ionized water at 20° C (68 °F). The cleaning- and disinfection solution is ready for use after 15 min.
3. Clean the products in the cleaning- and disinfection solution by using soft sponges. Areas difficult to access must be cleaned with soft brushes. Hard brushes and other materials which can damage the surface of the products must not be used.
4. Put the products into the cleaning- and disinfection solution with an exposure time of 15 min. Ensure complete wetting of all hollow spaces.
5. Remove the cleaning- and disinfection solution by thoroughly rinsing with de-ionized water. Remaining residues can shorten the product shelf life or can lead to material damage.
6. Dry the products. Accumulation of water must be avoided.
7. Check for visible contamination. If necessary repeat the reprocessing.
8. All parts must be dry and free of damage (cracks, ruptures etc.) prior to assembly. Assemble the product, as shown in the drawing "Assembly". Lubricate the seal ring with e.g. silicone spray prior to assembly.
9. Check the products acc. to the chapter "Control" and prepare them for sterilisation (see chapter "Packaging").

CONTROL

After disinfection and prior to each use, the products have to be checked for damages (cracks, ruptures etc.) and leaks. Damaged products must not be re-used.

PACKAGING

- Disassemble the product, as shown in the drawing "Assembly".

After the cleaning- and disinfection procedure the products must be packed together into a pouch which is suitable for steam sterilisation.

STERILISATION

Notes regarding sterilisation:

- Protect the products and sterilisation pouch against mechanical damage.
- Must not be connected to other connectors during sterilisation.

Sterilisation validated by the manufacturer:

- The components must be steam sterilised by a fractional vacuum process.
- Exposure time is 5 min with a sterilisation temperature of 134 °C (273 °F).

REUSABILITY

The end of the shelf life for reusable products is basically determined by wear and damage caused by its use. Considering the maximum shelf life of 5 years, reusable products can be reprocessed up to 100 times if appropriately used and under adherence of the cleaning-, disinfection- and sterilisation instructions.

Every further reuse is in the responsibility of the user (see chapter "Control").

WARNING

By using the products on patients with suspected prion disease there might be a high risk of transmission. In such case the physician decides if the product is either disposed (see chapter "Disposal") or reprocessed acc. to the national directives.

STORAGE CONDITIONS

Considering the maximum shelf life of 5 years, the product can be stored in its original packaging under the following conditions:

- Keep dry and cool.
- Keep away from sunlight and light sources.
- Don't store silicone products together with PVC- or rubber products.

SERVICE

Prior to return medical products for complaint/repair they must have run through the complete reprocessing process to exclude any risk for the manufacturer's staff. For safety reasons, the manufacturer preserves the right to refuse soiled or contaminated products.

DISPOSAL

Used or damaged products must be disposed acc. to applicable national and international legal directives.

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