## Tube-Tray

---

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## INTENDED USE

Tube-Tray is a tube holder for the fixation of adult and paediatric respiration tubing.

Clinical benefit: For strain relief of respiration tubing on the patient's face

Patient target group: Adults and children

Location of use: Rooms suitable for medical purposes

## INDICATIONS

- Fixation and strain relief of respiration tubing
- No other indications are known.

## CONTRAINDICATIONS

None known.

## SAFETY NOTE

- Read and follow the instructions for use carefully before using the product and keep them for future reference.

- This product must only be used by medically trained personnel.
- The user and/or patient must report all serious adverse events that occurred in connection with the product to the manufacturer and competent authorities of the EU Member State (or report to the competent authorities of the country if an event occurs outside of the EU) in which the user and/or patient is located.
- The products must be inspected visually for damages (cracks, breakage etc.). Damaged products must not be used.
- No changes may be made to the product.
- Prior to every use, the product must be disinfected (see Section "Wipe disinfection").

## VISUAL INSPECTION

- ▶ Check the product for damages and loose particles.
- ▶ Remove loose particles.

The faulty product must be disposed of (see Section "Disposal").

## USE

### CAUTION

- ▶ Push the Tube-Tray far enough under the support of the operating table so that there is no leverage effect.
- ▶ Hold the Tube-Tray during patient transfer to avoid tilting.
- ▶ Make sure that the Tube-Tray with rail clamp is firmly in place.
- ▶ When inserting the respirator tubing, make sure that it fits into the recess of the Tube-Tray and is not damaged in the process.

## REF 77-00-020

- ▶ Hang the Tube-Tray with the rail clamp on the standard rail of the operating table and fixate it with the screw.

## REF 77-10-000

- ▶ Push the Tube-Tray under the support of the operating table to ensure a secure hold.

## REF 77-20-000

- ▶ The respiration tubing can be fixed to the short or long bracket section. This allows the respiration tubing to be mounted at two different heights. Select the desired bracket section for fixation.
- ▶ Push the other bracket section under the support of the operating table to ensure a secure hold.

## WIPE DISINFECTION

### CAUTION

- The product may not be mechanically or manually reprocessed or sterilised.
- Do not immerse the product in liquids.
- If wipe disinfection is not successful, or if the product is strongly contaminated, the product must be disposed of as specified in Section "Disposal".

Wipe disinfection is to be performed with commercially available alcohol-based surface disinfectants. When choosing a product for disinfection, a disinfectant with appropriate ranges of action must be used: bactericidal, yeasticidal and virucidal. After wipe disinfection, inspect the product for visible contamination. If necessary, repeat wipe disinfection. After wipe disinfection, check the function of the product (see Section "Visual check").

## SHELF LIFE

The shelf life of the product is 5 years.

## STORAGE AND TRANSPORT CONDITIONS

### CAUTION

- Protect from heat and store in a dry place.
- Keep away from sunlight and light sources.
- Store and transport in the original packaging.

## SERVICE

Medical devices returned for complaint/repair must first be cleaned and disinfected in order to rule out any hazard for the manufacturer's staff. For safety reasons, the manufacturer reserves the right to refuse soiled or contaminated products.

## DISPOSAL

The used or defective products must be disposed of in accordance with the applicable national and international statutory regulations.

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