

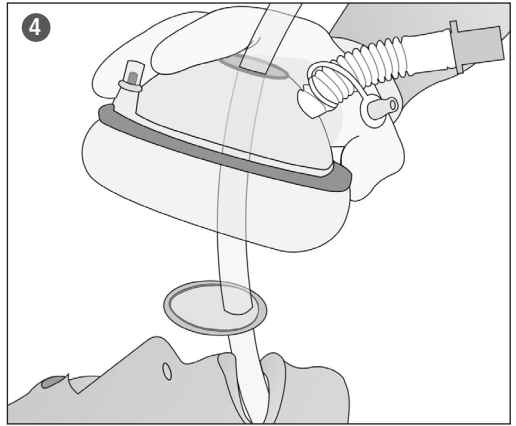
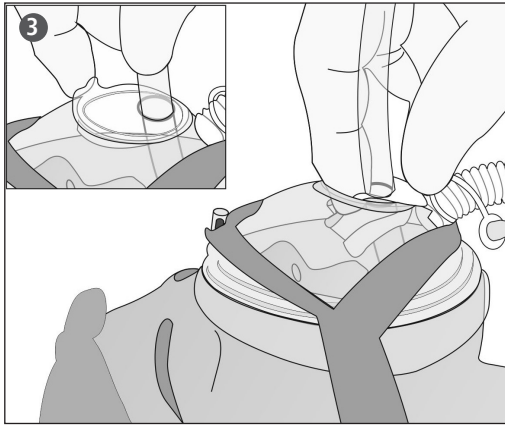
Endoscopy Mask

for single use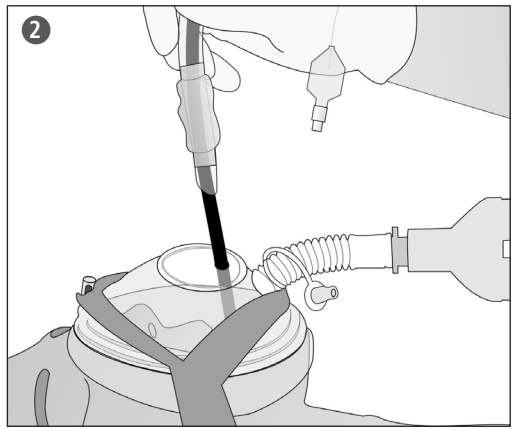

DE	ENDOSKOPIEMASKE Gebrauchsanweisung	4
EN	ENDOSCOPY MASK Instructions for use	4
DA	ENDOSKOPIMASKE Brugsanvisning	5
ES	MASCARILLA DE ENDOSCOPIA Instrucciones de uso	5
FR	MASQUE DE FIBROSCOPIE Manuel d'utilisation	6
IT	MASCHERA PER ENDOSCOPIA Istruzioni per l'uso	6
PT	MÁSCARA DE ENDOSCOPIA Instruções de utilização	7
SV	ENDOSKOPIMASK Bruksanvisning	7
TR	ENDOSKOPI MASKESI Kullanım kılavuzu	8

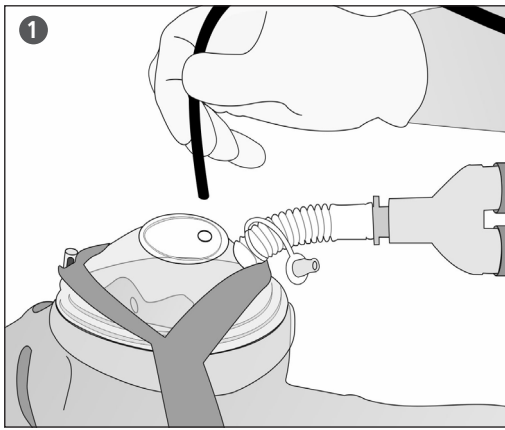Deutsch

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English

The cushion can be inflated and deflated with a commercially available syringe.

Warning

Before each use the products need to be checked carefully for damages (cracks, breakage, etc.). Damaged products must not be reused!

- 1 Ventilation of the patient through the Endoscopy Mask with supply of oxygen and anaesthetic gases. Connect the breathing system circuit directly to the 15 mm connector of the Mask, or to the included connecting tube. The membrane can be repositioned depending on oral or nasal intubation.
- 2 The fibrebronchoscope is introduced under sight through the hole of the membrane into the trachea. Afterwards the Endotracheal Tube is pushed along the fibrebronchoscope into the trachea. Lubricate the bronchoscope and Endotracheal Tube for easy insertion through the membrane.
- 3 Endotracheal Tube in situ. After taking off the fibrebronchoscope, the mask can be removed by pushing the membrane inside. This allows for the connector to stay on the Endotracheal Tube.
- 4 Connect the respiration tubing from the Endoscopy Mask to the Endotracheal Tube. The membrane stays in place with the Endotracheal Tube or can be cut off if necessary.

Attention

The Endoscopy Mask is for single use and must not be reused! Further recommendations for our single use products as well as storage are available at www.vbm-medical.de in the download section.

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