Manual Suction Pump



for quick suction of fluids

Intended to be left blank.

ASSEMBLY 4
DE HANDABSAUGPUMPE	Gebrauchsanweisung 5 - 6
EN MANUAL SUCTION PUMP	Instructions for use..... 6 - 7
BG РЪЧЕН АСПИРАТОР	Инструкции за употреба 8 - 9
CS RUČNÍ ODSÁVAČKA	Návod k použití..... 10 - 11
DA MANUEL SUGEPUMPE	Bruganvisning 11 - 12
EL ΧΕΙΡΟΚΙΝΗΤΗ ΑΝΤΛΙΑ ΑΝΑΡΡΟΦΗΣΗΣ	Οδηγίες χρήσης..... 13 - 14
ES BOMBA DE SUCCIÓN MANUAL	Instrucciones de uso 15 - 16
ET KÄSI-IMIPUMP	Kasutamisejuhised..... 16 - 17
FI KÄSIKÄYTTÖINEN IMUPUMPPU	Käyttöohje 18 - 19
FR POMPE D'ASPIRATION MANUELLE	Manuel d'utilisation 19 - 20
HR RUČNA USISNA PUMPA	Upute za uporabu 21 - 22
HU KÉZI ELSZÍVÓPUMPA	Használati utasítás 22 - 23
IT POMPA DI ASPIRAZIONE MANUALE	Istruzioni per l'uso..... 24 - 25
LT RANKINĖ ATSIURBIMO POMPA	Naudojimo instrukcija..... 25 - 26
LV MANUĀLĀS ATSŪKNĒŠANAS SŪKNIS	Lietošanas instrukcija 27 - 28
NL HANDAFZUIGPOMP	Gebruiksaanwijzing 28 - 29
NO MANUELL SUGEPUMPE	Bruksanvisning..... 30 - 31
PL POMPA RĘCZNA DO ODSYSANIA	Instrukcja użycia..... 31 - 32
PT BOMBA DE ASPIRAÇÃO MANUAL	Instruções de utilização 33 - 34
RO POMPA MANUALĂ DE ASPIRAȚIE	Instrucțiuni de utilizare 34 - 35
RU РУЧНОЙ АСПИРАТОР	Инструкция по применению 36 - 37
SK RUČNÁ ODSÁVAČKA	Návod na použitie 38 - 39
SL ROČNA SUKCIJSKA ČRPALKA	Navodila za uporabo 39 - 40
SV HANDSUGPUMP	Bruksanvisning..... 41 - 42
TR MANUEL EL POMPASI	Kullanım kılavuzu 42 - 43
SYMBOL DESCRIPTION 44 - 45

ASSEMBLY

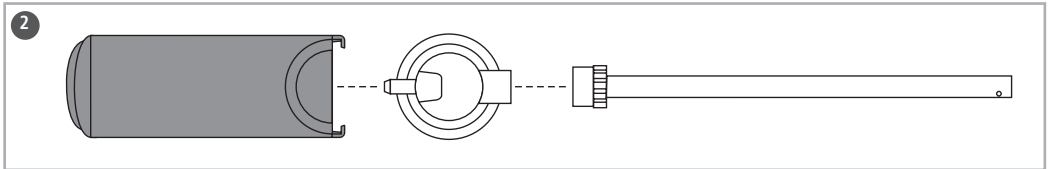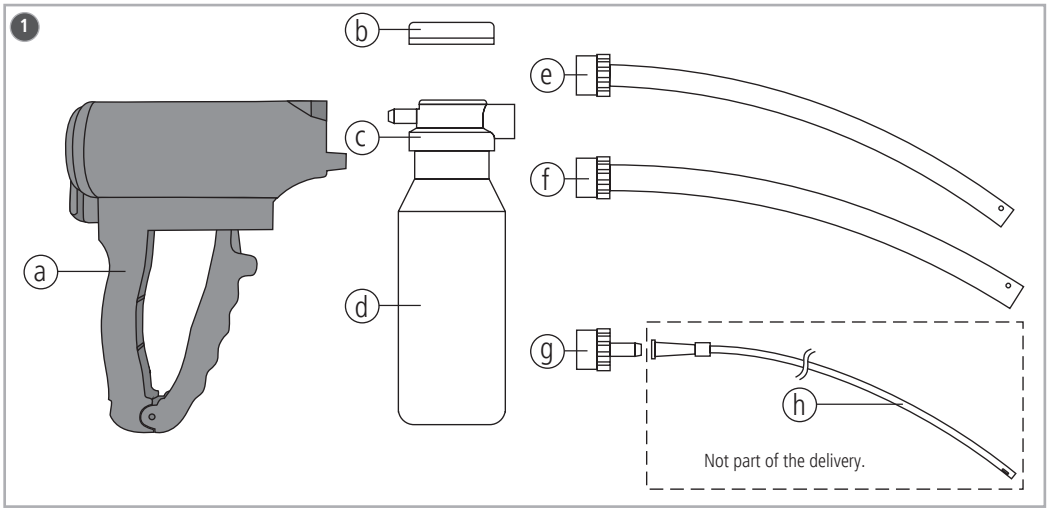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English

INTENDED USE

The Manual Suction Pump is used to provide a quick and effective suction of fluids from the oral cavity and the pharynx. The accessories are used for suctioning and collecting the fluids.

Clinical benefit: Reduces the risk to aspirate fluids from the oral cavity and the pharynx.

Patient target group: Patients who require suctioning of fluids from the oral cavity and the pharynx.

Place of use: hospital and pre-hospital conditions

INDICATION

- Aspiration of fluids from the oral cavity and the pharynx.

Further indications are not known.

CONTRAINDICATION

None known.

SAFETY INSTRUCTIONS

- Read and follow the instructions for use carefully before using the product.
- The product may only be used by medically trained personnel who have sufficient knowledge in handling the product.
- The user and/or patient must report all serious adverse events that occurred in connection with the product to the manufacturer and competent authorities of the EU member state (or report to the competent authorities of the respective country if an event occurs outside of the EU) in which the user and/or patient is located.
- The product is delivered ready to use. The vacuum pump must be cleaned thoroughly after every use (see "Wipe disinfection"). Damaged products must not be used.
- Maximum application time: 60 minutes
- No changes may be made to the product.
- Use original VBM accessories only. Using accessories from other manufacturers may have a negative impact on the function of the medical device. The manufacturer accepts no liability for this. Exception: Standard suction catheters are permitted.
- The product is not suitable for MRI.

- The product is not sterile.
- The Accessory Kit for Manual Suction Pump is intended for single use and must not be reused and/or reprocessed. Reprocessing influences the function of the Accessory Kit for Manual Suction Pump. Any reuse entails a potential infection hazard.

PRODUCT DESCRIPTION (FIG. 1)

a	Vacuum pump	Reusable
b	Sealing cap	Accessory Kit for Manual Suction Pump For single use
c	Suction cap	
d	Container 250 ml	
e	Suction catheter 28 Fr	
f	Suction catheter 40 Fr	
g	Adapter for standard suction catheter	
h	Standard suction catheter	

PREPARATION PRIOR TO USE

- ▶ Connect the container including the suction cap to the vacuum pump (Fig. 2).

VISUAL AND FUNCTIONAL CHECK

- ▶ Check the product for damages and loose particles.
- Before using the device, check whether a vacuum is generated:
- ▶ Seal the opening of the suction cap with your thumb or palm of your hand.
 - ▶ Operate the vacuum pump handle several times.

This allows you to determine if a vacuum is generated.

The product must not be used if it did not pass the visual and functional check.

USE

- ▶ Select the suction catheter according to the use and connect it to the suction cap (Fig. 2).
- ▶ Alternative: Connect the adapter for standard suction catheter to the suction cap. Then connect a standard suction catheter.

CAUTION

- ▶ Carefully insert the suction catheter into the patient's oral cavity and pharynx.

To prevent fluids from contaminating or obstructing the vacuum pump, the container must be kept as upright as possible.

- ▶ Operate the vacuum pump handle.

CAUTION

The container may be filled with max. 250 ml.

CAUTION

If the vacuum pump can only be operated with difficulty, it should be checked whether there is an obstruction.

- ▶ Do not continue to suction until the obstruction has been eliminated.

- ▶ As soon as suctioning is complete, carefully remove the suction catheter from the patient's oral cavity or pharynx.
- ▶ Disconnect the filled container from the vacuum pump.
- ▶ Disconnect the suction catheter from the suction cap.
- ▶ Unscrew the suction cap from the container and cover with the sealing cap.
- ▶ If a lab test is required, the sticker on the container must be labelled.
- ▶ The used Accessory Kit for Manual Suction Pump must be disposed of completely (see "Disposal") and replaced with a new Accessory Kit for Manual Suction Pump REF 30-56-000.

WIPE DISINFECTION

CAUTION

- The vacuum pump must not be mechanically or manually reprocessed or sterilised.
- If wipe disinfection is not successful, or if the product is strongly contaminated, the vacuum pump must be disposed of as specified in section "Disposal".

Wipe disinfection must be performed with commercially available surface disinfectants based on alcohol, QACs (quaternary ammonium compounds) or hydrogen peroxide. When choosing a product for disinfection, a disinfectant with appropriate ranges of action must be used: bactericidal, yeasticidal and virucidal. After wipe disinfection, inspect the vacuum pump for visible contamination. If necessary, repeat wipe disinfection. After wipe disinfection, check the function of the vacuum pump (see "Visual and functional check").

SHELF LIFE

Vacuum pump

The shelf life of the product is 5 years.

Accessory Kit for Manual Suction Pump

Use-by date: See product label

STORAGE AND TRANSPORT CONDITIONS

CAUTION

- Protect from heat and store in a dry place.
- Keep away from sunlight and light sources.
- Store and transport in the original packaging.

SERVICE

Prior to the return of medical devices for complaints/repairs, the products must be cleaned to exclude any risk for the manufacturer's staff. For safety reasons, the manufacturer reserves the right to refuse soiled or contaminated products.

DISPOSAL

The used or damaged products must be disposed of in accordance with the applicable national and international statutory regulations.

PRODUCT SPECIFICATIONS

Max. achievable vacuum	480 mmHg
Vacuum pump dimensions	155 x 55 x 180 mm
Weight REF 30-55-000 (incl. packaging)	470 g

SCOPE OF DELIVERY

REF	Description	Packaging unit
30-55-000	<ul style="list-style-type: none">• Vacuum pump• Accessory Kit for Manual Suction Pump	1 pc

CONSUMABLES

REF	Description	Packaging unit
30-56-000	Accessory Kit for Manual Suction Pump	1 pc

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